

“Camera+ legacy” settings

To help you get the best images

Camera + Legacy

Latentite apps

iOS only

Use on older iPhones which cannot
run procamera or Camera+ 2

Open settings

select file format

Chose JPG for the best compatibility, TIFFs, HEICs and RAW DNG images are not always viewable without specific software .

Chose High quality for JPG- this will increase file size but give you the best quality JPG image .

Chose JPG and RAW DNG- this will save two copies of the image , one in RAW DNG and one in JPG format - this will take lots of space but will allow you the best image quality of post processing of the image is required

general settings

- 1) Save to camera roll this ensures image will auto upload to google photos and If you delete app you don't delete all you photos (I've don't this several times !)
- 2) Turn Geotagging on - this allows you to search on location of image at a later date
- 3) Volume buttons on - this allows you to take picture by pressing volume buttons- really helps when holding phone at an odd angle

set up manual controls

Select :

- 1) Exposure circle turned on - this will allow you to tap to select exposure based on a part of the image- essential for retinal photography to avoid tapetal overexposure
- 2) manual focus on
- 3) Always active on - swipe vertically at any point on the image to manually focus
- 4) Focus peaking on- this is really useful for ensuring fine focus on difficult to see structures like retinal vessels

Light controls

- 1) Tap top left corner (red circle)
select “torch mode”
- 2) light intensity needs to be
reduced by placing micropore
tape or medicine labels over the
LED